


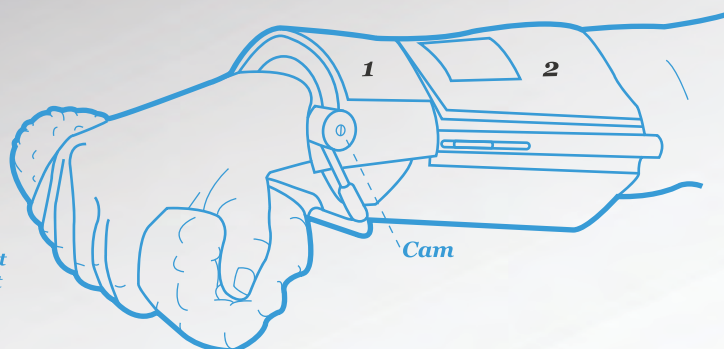


FITTING INSTRUCTIONS FOR THE WRIST EXTENSION

1. Open strap #1 and strap #2.
2. Position patient's hand, wrist and forearm inside.
3. Align patient's wrist joint between cams.
4. Close strap #1 across top of wrist.
5. Close strap #2 across top of forearm.
6. Secure hand inside of hand piece closing strap #3 and strap #4 across fingers, when applicable.


*Padded
Palmar
Hand/Wrist
Attachment*



*Anti-
Spasticity
Ball Hand/
Wrist
Attachment*

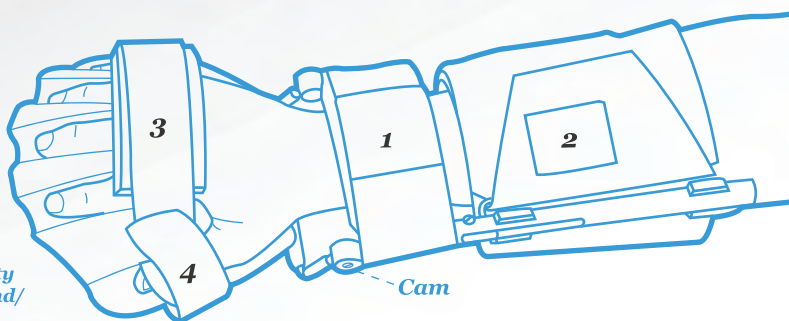

*Mitt
Hand/Wrist
Attachment*

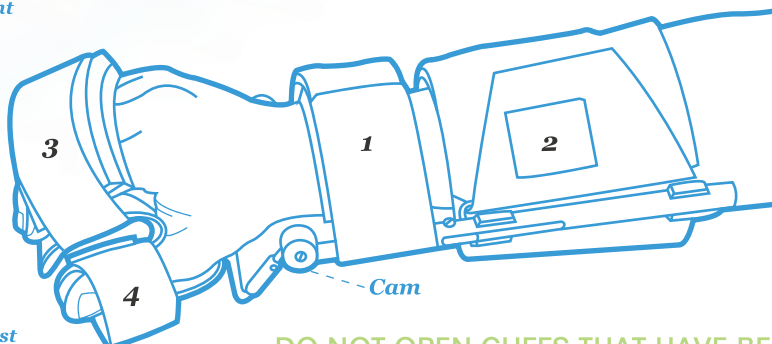

DO NOT OPEN CUFFS THAT HAVE BEEN SECURED WITH DUCT TAPE OR GLUE